


CENTURION®

WingGuard®

PICC SECUREMENT DEVICE

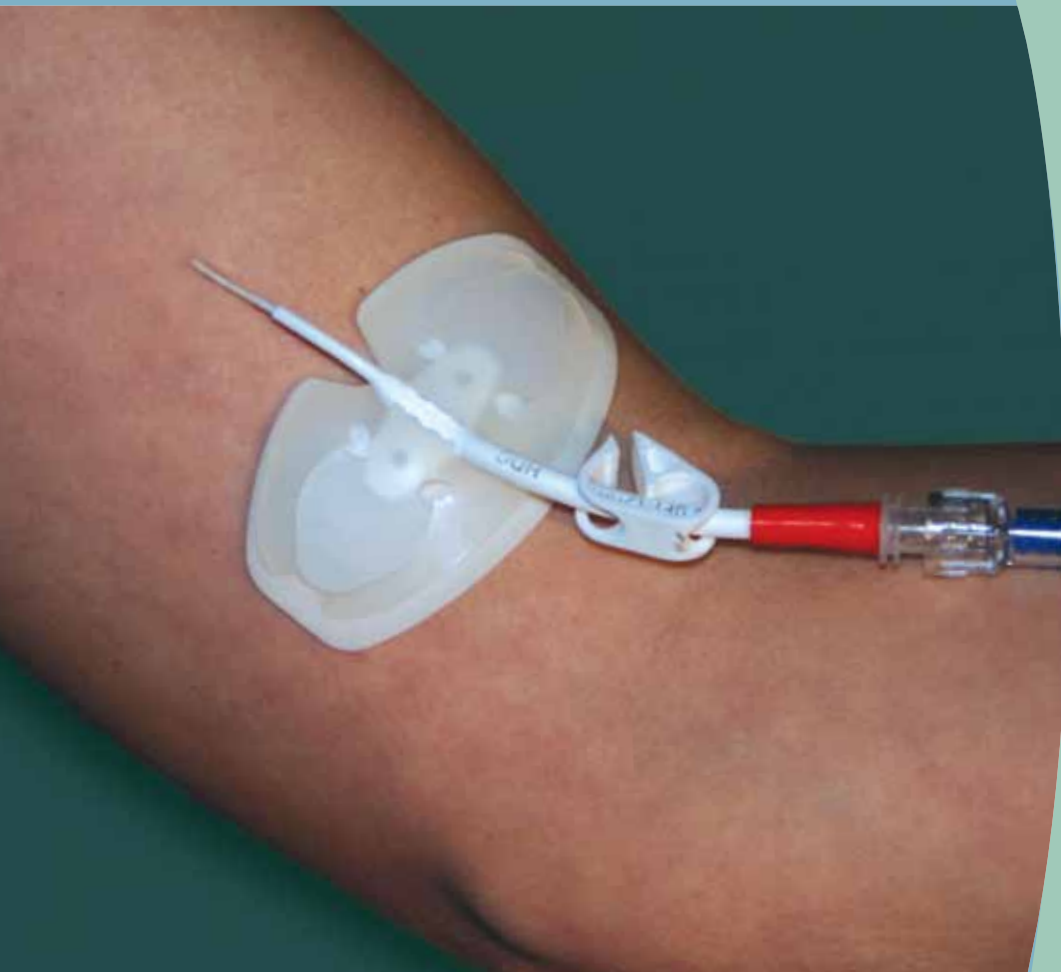

patient friendly

nurse friendly

dressing friendly

Universal device: one size secures virtually any size or type of PICC catheter

Pliable construction conforms comfortably to body contours

Low profile design minimizes bumping and jostling to site and improves dressing adherence

Silicone top layer allows for clean removal of dressings

Extremely easy to apply and to remove

Patient-friendly adhesive is gentle, yet stays in place for long wear times

Available in two latex-free materials

"Improved adhesive products and securement devices... are essential to provide increased catheter stability.

Such products... may reduce catheter dislodgment and the necessity of reinsertion with its associated needlestick risk."

- from the OSHA* *FactSheet: Securing Medical Catheters*

WingGuard® meets INS Standards for Catheter Stabilization and Manufactured Stabilization Devices

Patent Pending







Available packaged individually sterile or as part of complete Dressing Change or Securement Kits

WingGuard®

PICC SECUREMENT DEVICE

APPLICATION DIRECTIONS

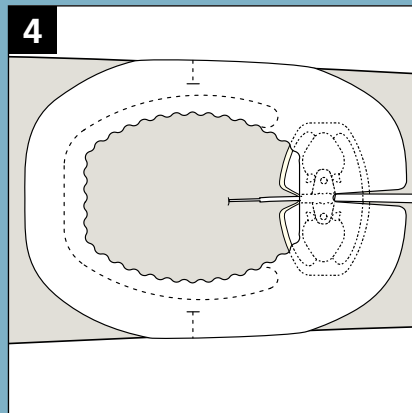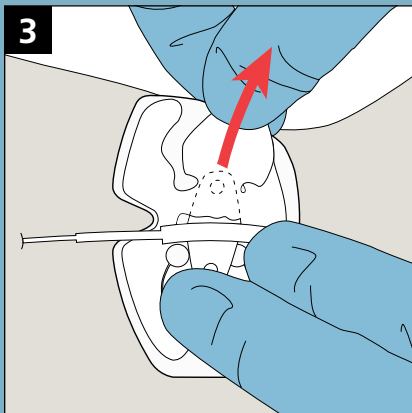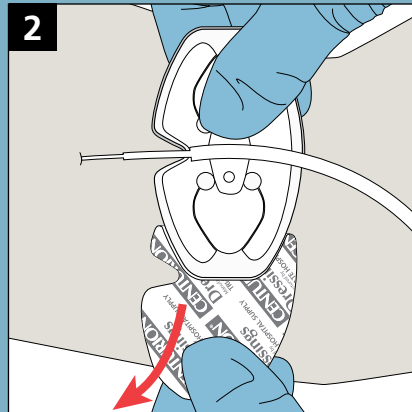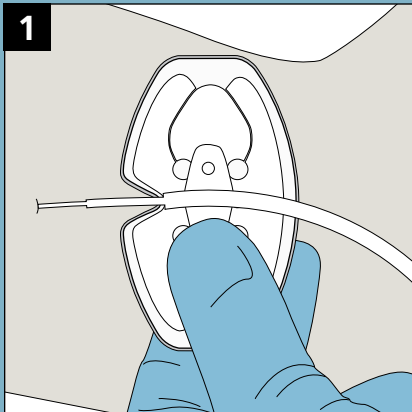

APPLICATION

Thoroughly clean and prepare site. Allow to dry completely.

1. Slide WingGuard under catheter hub and center hub wings over WingGuard.

2. Holding catheter in place, peel off half of liner. Press to adhere to skin.

Repeat on other side.

3. Holding catheter in place, pull tab out from center to create an opening in slit. Insert first catheter wing into slit.

Repeat on other side.

4. Apply preferred dressing over the site as usual.

REMOVAL

1. Remove dressing.

Optional: Apply sterile tape strip to catheter tubing to help stabilize catheter prior to removing WingGuard.

2. While stabilizing the catheter, pull WingGuard tab out from center to create an opening in slit to release catheter hub wings from device.

3. Peel WingGuard slowly off skin, away from exit site.